
A Complete Workflow for the Disinfection of Ultrasound Probes using

UV Smart D45




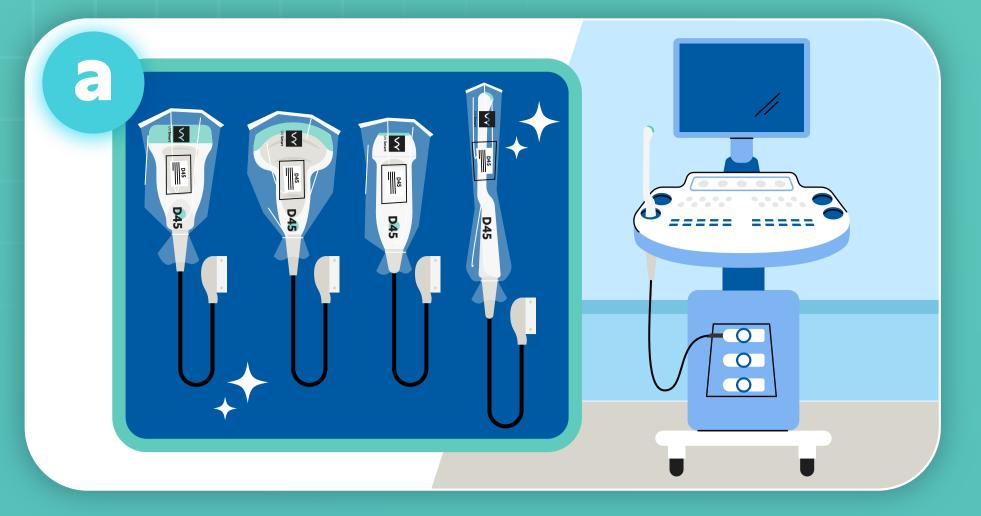


Remove the probe cover from the ultrasound probe you intend to use and connect it to the echocart.



Disinfect hands, wear gloves, and prepare the probe.

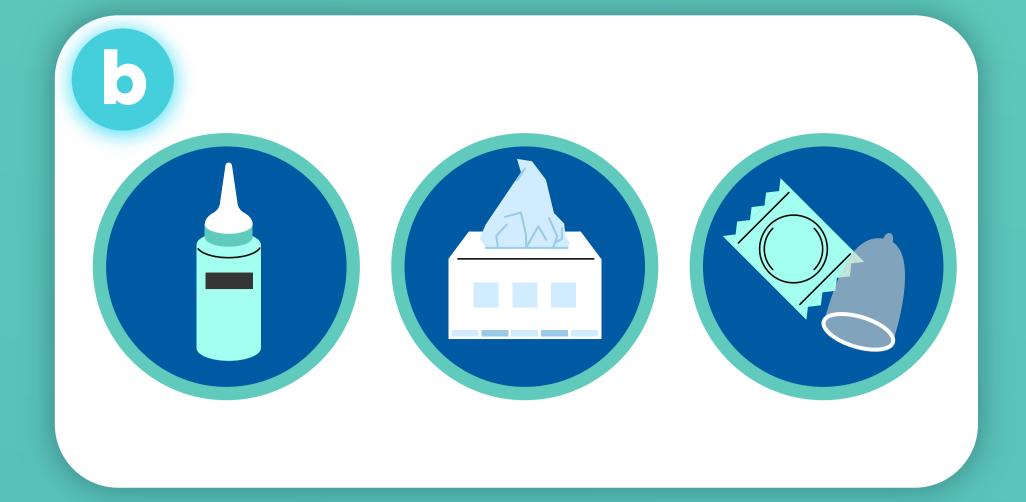

In the room, ensure availability of ultrasound gel, microfiber wipes and sheaths.

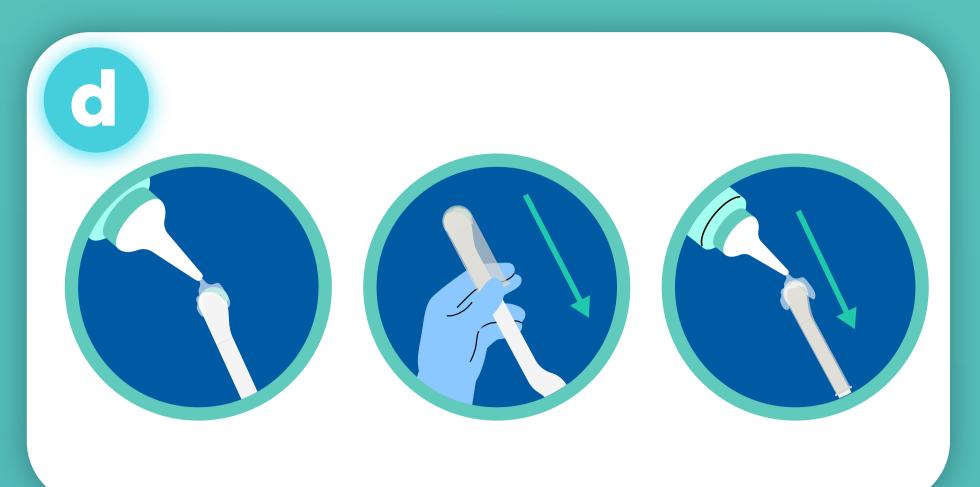


Follow the gel-sheath-gel process for transvaginal probes.

2 Solution **Examination**






Perform ultrasound scan with proper hand hygiene.

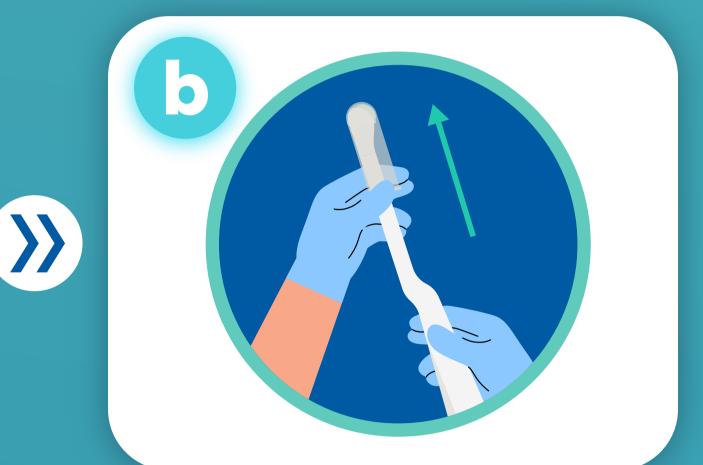

Remove the sheath.

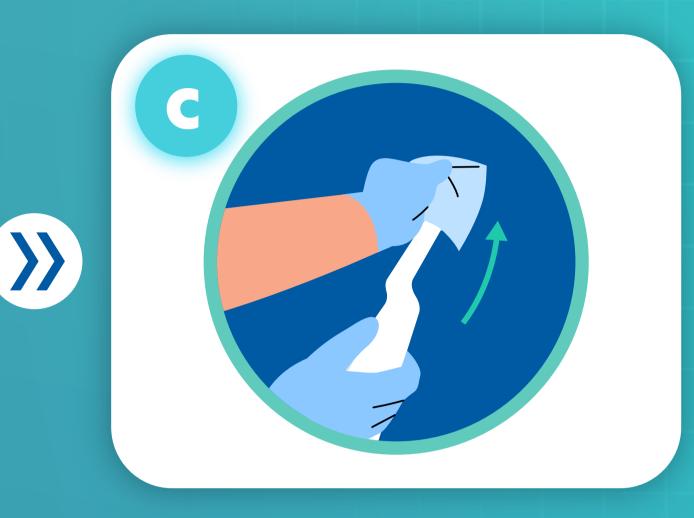

Clean residues/gel from the probe tip/nose.









The D45 prompts remind the user to clean the device. Regularly perform a visual check and clean the device according to the IFU. <image>

Clean the internal and external surfaces of the D45 with a microfiber wipe and UV Smart Device Cleaner.





Validation Sticker



Validation can also take place every 48 hours, while cleaning the D45, using validation stickers.



Stick the validation stickers in the booklet for validation traceability.